


Pålægning af løftesejlet

Sæt stropperne fra løftesejlet på krogene på løftebøjlen. Sejlets bagerste stropper placeres på de kroge, der er vendt mod brugeren, og sejlets benstropper placeres på de kroge, der vender væk fra brugeren.

Advarsel!

Vær opmærksom når du placerer sejlstropperne på løftebøjlen. Kontroller at sejlstropperne er placeret korrekt i krogen på løftebøjlen. Når der trykkes på pil op knappen på håndbetjeningen check igen at sejlstropperne forbliver i den korrekte position i løftebøjles krog. (Fig. 1).

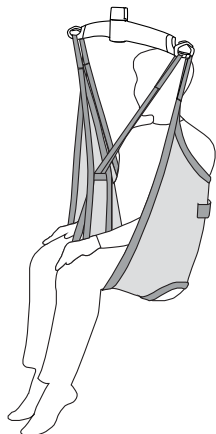

Attaching the lifting sling

Place the straps from the lifting sling on the hooks of the lifting hanger. Attach the uppermost set of straps (from the back) to the hooks that face the user. Attach the lowermost set of straps (from the legs) to the hooks that face away from the user.

Caution!

Be careful when attaching the lifting sling's straps on the hooks. Check that the straps have been correctly placed in the lifting hanger's hooks. When pressing the up button on the hand control to lift the user, check again that all straps remain correctly placed in the lifting hanger's hooks (Fig. 1).

