



Quick Guide - Combi-lock



Position the traverse rail so it is aligned with the Combi-lock rail.




When the rails are aligned the safety gates open automatically.



After checking that the Safety lock has engaged, position yourself right behind or in front of the hoist. It is now safe to manoeuvre the hoist through the Combi-lock, and on to the fixed or traverse rail.



When the hoist has passed through the Combi-lock it is safe to move the traverse rail.

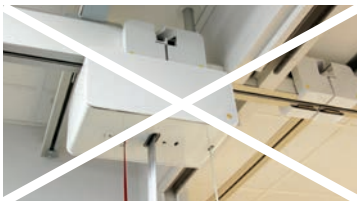

Note: Do not move the traverse rail when the hoist is in this position!



Quick Guide - Combi-lock



Place the hoist at the yellow marking approx. 1 meter from the end of the traverse rail and Combi Lock.



Place the patient facing / looking lengthwise the traverse rail and in a level where you can push / hold as in the picture.

To align the rails and engage the Combi Lock place yourself on the side of the patient so you push or pull perpendicular on the traverse rail.

If the Combi Lock doesn't engage the first time, move the traverse rail approx. ½ to 1 meter away from the Combi Lock rail and try again.



When the Combi Lock has engaged position yourself behind the patient and push lengthwise the traverse rail through the Combi Lock. The patient is still facing / looking lengthwise the traverse rail.

taylordolman

tel: 0845 838 0600
email: info@taylordolman.com
www.taylordolman.com