DK . . . .	Drejeskive for gulv . . . . .	2
GB . . . .	Turning Disc for use on floor . . . . .	5
D . . . . .	Drehscheibe für Boden . . . . .	8
S . . . . .	Vridskiva för golv . . . . .	11
U.S.A. and countries outside the EU . . . . .		14

Vers. 1.00

**taylordolman**

tel: 0845 838 0600  
email: [info@taylordolman.com](mailto:info@taylordolman.com)  
[www.taylordolman.com](http://www.taylordolman.com)

Guldmann™

# DK . . . . . Drejeskive for gulv

Varenummer:

50010

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---

### 1.02 Important/Warning

- Read the user manual fully before using the turning disc.
  - Do not use the turning disc for other purposes than those indicated in the manual.
  - Do **not** exceed the maximum load of 200 kg.
  - Do not use the turning disc under water, for instance in bath tubs etc.
  - Do not leave the turning disc in walking areas such as corridors, halls, etc.
  - Do not use the turning disc for persons with an unstable standing function.
- 

---

## 2.00 Description of Function

---

### 2.01 Operation

#### Conditions for using the turning disc

The user must be able to keep his balance in a sitting and standing position  
The carer must be able to support the user in an appropriate way to eliminate the risk that the user should fall.

If possible, use the Max Transfer Band from Guldmann together with the turning disc to ensure a safe and good support.

#### Procedure

1. Assist the user to an upright sitting position in the chair, wheel chair, bed or toilet, so that his feet touch the floor.
  2. Place the turning disc under the user's feet, the handle of the turning disc pointing to the user.
  3. Place the foot/feet in the middle of the turning disc.
  4. Raise the user to a standing position using a technique agreed upon between the user and carer.
  5. Rotate the user to the correct position by supporting the hip.
- 

### 3.00 Transport

Use the handle to transport/carry the turning disc.

---

---

**4.00 Maintenance and storage**

---

**4.01 Cleaning**

Wipe the turning disc using a cloth wrung in ordinary soapy water or clean it with cleaning alcohol.

---

**4.02 Storage**

Do not leave the turning disc in walking areas after use.

Do not hang the turning disc close to a source of heat or directly in the sun as the lubricant of the ball bearings may run out.

---

---

**5.00 Technical specifications**

Dimension (WxHxL) . . . . . 325x15x375 mm

Maximum lifting capacity . . . . . 200 kg

Guarantee: . . . . . 1 year

**The guarantee does not cover if**

- There are visible signs that the two plates have been twisted apart with the result that the balls have fallen out.
- The turning disc shows breakage or other damage after having been used for other purposes than those indicated in the manual.

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## U.S.A. and countries outside the EU

### A. **USERS GUIDE**

Read the entire warranties and the entire chapter before use of the product.

### B. **WARRANTIES**

Guldmann equipment is designed to be used for transferring persons with a physical handicap who are not able to physically selftransfer.

This equipment is designed to relieve the users, their relatives, and nursing personnel in the task of transferring elderly or handicapped people, thereby minimizing the risk of back injury during such lifting and transfer operations.

If the product is used in any fashion or in an irresponsible manner or for any use other than that described above or in the enclosed written instructions, the manufacturer's warranty will be nullified and rendered void.

There are no warranties which extend beyond the description in the enclosed written instructions.

Guldmann warrants that its equipment is free from defects in materials and workmanship under normal use. Guldmann warrants that the equipment will perform substantially in accordance with the specifications set forth in the documentation provided with the equipment.

The above express warranties are made for a period of 365 days from the date the equipment is delivered to you as the first user.

Your distributor will replace any equipment which proves defective in materials or workmanship, without additional charge, on an exchange basis.

Your distributor will either replace or repair without additional charge any Guldmann equipment that does not perform in substantial accordance with the specifications of the documentation.

Guldmann does not warrant that the functions contained in the devices will meet your requirements. The warranty does not cover any of the part of the equipment which has been subject to damage or abuse by you. The warranty does not cover any part of the equipment which has been altered or changed in any way by you or others.

Any implied warranties including any warranties of merchant ability or fitness for a particular purpose are limited to the term of the express warranties.

Guldmann shall not in any case be liable for special, incidental, consequential, indirect or other similar damages arising from any breach of these warranties even if Guldmann or its agent has been advised of the possibility of such damages.

You must call Guldmann or your distributor for an authorization to return any defective item during the warranty period. If your distributor is unable to correct your problem by telephone, you will be provided with a return authorization number and address for returning the defective item for warranty service or replacement. You must insure any defective item being returned because Guldmann does not assume the risk of loss or damage while in transit. Do not return items or warranty service to Guldmann.

The warranties set forth above are in lieu of all other express and implied warranties, whether oral, written or implied, and the remedies set forth above are your sole and exclusive remedies. Only an authorized officer of Guldmann may make modifications to this warranty, or additional warranties binding on Guldmann. Accordingly, additional statements such as advertising or presentations, whether oral or written, do not constitute warranties by Guldmann and should not be relied upon as such. The warranty gives you specific legal rights, and you may also have the other rights which vary from state/country to state/country.

| Time to care |

**taylordolman**

tel: 0845 838 0600

email: [info@taylordolman.com](mailto:info@taylordolman.com)

[www.taylordolman.com](http://www.taylordolman.com)