



THE BATHROOM OF THE FUTURE

- FOR PEOPLE WITH DISABILITIES

FOR A SAFER AND MORE
INDEPENDENT LIFE

taylordolman

ROPOX™
BETTER WAYS TO BETTER DAYS



MAKING IT EASIER TO TAKE CARE OF YOURSELF

The bathroom is a place we use several times a day, and it must be safe, comfortable, and easy to use – also if you need extra support. In a flexible bathroom, aids and assistive devices can be adjusted to meet individual needs, and this makes it an optimal solution for both users and their helpers.

As we get older, our physique weakens, and gradually, we need more and more aids to be independent. With a flexible bathroom, we can use the same decor for a longer time because the products can be adjusted to match our changing abilities and the different needs of several users, which makes it an ideal solution for hospitals, rehabilitation centres as well as in private homes.

A bathroom furnished with flexible aids and assistive technology does not only make the user more independent, but it also offers the helper ideal working conditions. As it is a long-lasting and cost-effective solution, it is no doubt worth the money.



CONTENTS

- 04 A BATHROOM THAT MEETS YOUR NEEDS
The right design makes everyday life better
- 06 WASHBASINS
For an easier life
- 08 TOILET LIFTER AND TOILET SUPPORTS
Aids that make you more independent
- 10 BATHROOM SOLUTIONS
Safe bathing and showering
- 12 A SAFE BATHROOM
For the bathroom interior - feeling comfortable and supported
- 14 BATHROOM DESIGN IDEAS
Different needs require different solutions

With the right aids,
you can do much more
without help from others

taylordolman

tel: 0845 838 0600
email: info@taylordolman.com
www.taylordolman.com

ROPOX™
BETTER WAYS TO BETTER DAYS



A BATHROOM THAT MEETS YOUR NEEDS

THE IDEAL SOLUTION FOR WHEELCHAIR USERS

For a wheelchair user, personal care is not always easy, but with the right aids you can become more independent and get a higher quality of life.

A swing washbasin makes it much simpler to use the toilet because it can rotate and swing 180° so that you can place it in front of the toilet. This way you can wash your hands while seated on the toilet and minimise the transfers to the washbasin.

The swing washbasin is probably the most user-friendly basin for wheelchair users because there is enough room for the chair to get under it. The washbasin provides the best possible turning area for a wheelchair when pushing the washbasin against the wall.

It is essential that the bathroom décor and design meet the user's individual needs and physical condition if the room should be comfortable and easy to use.



THE BEST SOLUTION FOR WALKING-IMPAIRED

If you are walking-impaired, your bathroom products must be stable so that you can use them for support when moving around in the room. When you furnish a bathroom, it is essential to have as many support points as possible to reduce the risk of falling injuries.

A modern solution makes it easier to use the bathroom, and it also minimises the number of transfers. For example, you can adjust the support washbasin both in height and sideways, this way it can also serve as a supportive handle on the way to the toilet. Or maybe all you need is a pair of toilet support arms and a toilet lifter to be able to go to the toilet without a helper by your side.

With the right assistive technology, you feel safer in the bathroom, and you will be able to do more on your own.

A man with glasses and a dark sweater is seated on a toilet. He is leaning forward, using a built-in washbasin that is part of the toilet fixture. The washbasin has a white faucet and a white soap dispenser. A blue cloth is hanging from the side of the basin. The background is a light blue wall.

WASHBASINS

FOR AN EASIER LIFE

A washbasin must be easy to use, but it should also serve as a support for the user. That is what we keep in mind when we design all our basins: They must be user-friendly and safe.

To make them as user-friendly as possible, they can all be adjusted to meet individual needs, e.g. they are height-adjustable and some can be rotated out in the room so that you can reach them from the toilet if you want to wash your hands while seated.

With our assistive products, you become more independent, and there is of course also the benefit that you will get a stylish bathroom interior that is easy to clean.



Basic



Manual



Electric



Solid support grabs
for a safe movement

ROPOX

SwingLine



- Primarily suitable for wheelchair users
- The design provides sufficient legroom under the washbasin
- Suitable for sitting users, e.g. when washing hands, brushing teeth etc.
- Its ability to rotate and swing makes the washbasin accessible from all sides
- Can be adjusted in the height to meet individual needs
- Cleaning-friendly design without dirt traps
- Dock-in function is recommended to fasten the washbasin

ROPOX

SupportLine



- Especially suitable for walking-impaired
- The integrated, ergonomic grab rails can be used as a support to prevent falling injuries
- Can be height adjusted to meet individual user's needs
- Sideways slidable to minimise the distance when transferring to the washbasin
- The design provides sufficient legroom under the washbasin
- Simple and cleaning-friendly design minimises bacterial growth

ROPOX

Other washbasins



VanityLine*

- Sufficient legroom for wheelchair users
- Can be adjusted in height to match the individual user's needs
- Storage space on the washbasin
- Cleaning-friendly design
- Choose between standard, support or hospital washbasin**
- Unit with or without washbasin**
- Integrated safety stop*



StandardLine**
with hospital washbasin



AdaptLine**
with standard washbasin



SlimLine**
with support washbasin



TOILET LIFTER AND TOILET SUPPORTS

AIDS THAT MAKE YOU MORE INDEPENDENT

Most people want to be independent and handle their daily activities on their own – especially when it comes to personal care and hygiene. However, this may not be possible for people with physical or mental disabilities.

With the right aids and assistive products, you get the support you need when going to the toilet without a helper's assistance. Perhaps, a pair of toilet support arms is all that is needed to get up and reach a standing position; and with a toilet lifter, you can be lowered and lifted when getting up from or sitting down on the toilet. The best part is that it is done very gently.






ROPOX

Toilet Lifter


- Lifts and lowers the user to a sitting or standing position
- Easy height-adjustment for a much easier access to the toilet
- Provides the ideal sitting position on the toilet
- The ergonomic design with rounded edges makes them easy to clean and hygienic to use
- Integrated height adjustable toilet support arms

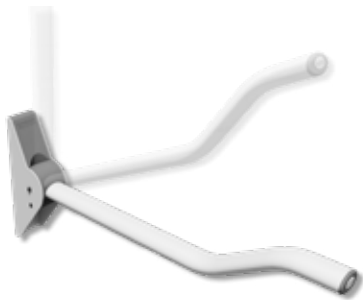

ROPOX

Toilet Support Arms

- Offer support and safety when moving around
- Can be adjusted in various positions
- The oval shape relieves the pressure on your hands and forearms
- Height adjustable and with two integrated heights - optimal support when seated and standing
- Cleaning-friendly – the smooth surface is easy to clean

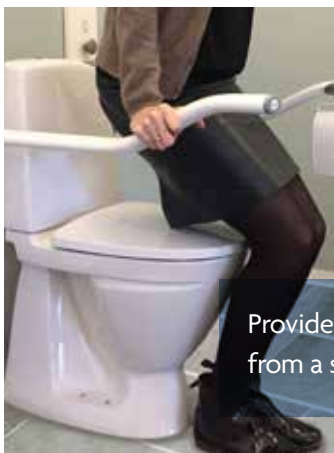

Provide support at several levels – from a sitting to a standing position.



The standing/sitting attachments provides further support when getting up.



Make it easy to get on and off the toilet.



BATHROOM SOLUTIONS

SAFE BATHING AND SHOWERING

For people with mental or physical disabilities, the combination of water, soap, and slippery tiles may feel uncomfortable and unsafe, and it may cause an increased risk of falling injuries and accidents. That is why you need solid and stable solutions that make the bathroom safe so that you can feel safe as well.

The solid grabs and shower rails offer a stable support, and if you need to rest, the toilet seat offers great support as well – even in a cramped place. When furnishing and designing your bathroom with the right aids and assistive products, you can reduce the need for assistance when showering, and you get the freedom to be more independent.





ROPOX

Shower seat


- Reduces the risk of falling accidents
- Height adjustable option
- Non-slip seat with a large surface and a solid grabbing edge
- Hard-wearing, insulating material which does not feel cold on your skin
- Good padding which makes the seat comfortable to sit on and prevents pressure sores
- Armrests and seat can be folded up to a vertical position leaving more space in the room



ROPOX

Grab and shower rails

- Several sizes and designs
- Provide a solid grip and an excellent support
- Cleaning-friendly design for a better hygiene
- A shower shelf can be positioned under the shower rail so that you still have a good hold on the rail

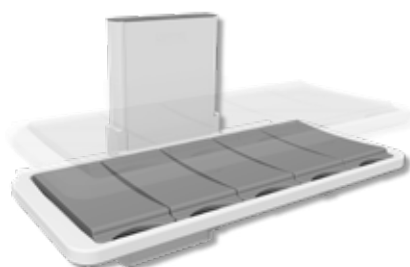

ROPOX

Shower/changing bed

- Height adjustment to meet the user's individual needs
- The slats have a soft top side, which makes them very comfortable to lie on and a flatter side for easier rolling
- Easy up and down adjustment by using a hand control
- Can be folded up to provide extra space in the room
- Cleaning-friendly as the slats are very easy to remove
- Back support can be mounted



ROPOX

Bathtub

- Adjustable in height to meet the user's individual needs
- Getting in and out of the bathtub can be adjusted depending on the person's physical condition
- By adjusting the height of the bathtub, a person can easily be lifted to the bathtub using assistive devices
- Thanks to the hand control, the bathtub is easy to height adjust
- Can be provided with a shower bed cover

Seated showering reduces the risk of falling injuries



A SAFE BATHROOM

FOR THE BATHROOM INTERIOR

Paint an area behind the bathroom furniture and cabinets in a contrasting colour (max. 2 different colours), in a luminance contrast of 30% to make it easier for visually impaired to see the white elements.

Choose a flooring material in a light colour. A floor with just one solid colour makes it easier for people with dementia to orientate themselves in the room because they may perceive dark spots on the floor as holes.

Use anti-slip linoleum on the floor. That makes the room safer - even when the floor is wet.

Choose a sliding door for wheelchair users. It is easier for them to get in and out, it will also provide a larger turning area in the bathroom. Choose a traditional door for people with dementia. They may have difficulties using a sliding door.

Paint the door in another colour than the walls. It is easier for visually impaired and people with dementia to see the door.

■ SUPPORT

■ ADJUST-
MENT

FEELING COMFORTABLE AND SUPPORTED

It is very important to furnish your bathroom with many support points and the right aids and assistive products, but other aspects should be considered as well:

- Do you need grab and shower rails in the bathroom for support?
- Do you need a height adjustable toilet and washbasin?
- Are you able to shower in a standing position, or do you need a shower seat?
- Do you need toilet supports at the toilet?
- Do you need more assistance than toilet supports to sit down on and get up from the toilet?
- Would it be an advantage if you could wash your hands while seated on the toilet so you do not have to move to the washbasin?
- Is the distance between the products minimised to reduce the number of lifts and movements?
- How do you ensure optimal space for a wheelchair user?


When the bathroom is furnished with flexible and user-friendly aids, you will feel safe, and when you feel safe, you are often able to do more things on your own. For many people, that is the essence of quality of life, and we are very proud to contribute to that.

In our bathroom concept video you will find more inspiration – please find it on ropox.com under the menu Bathroom concepts.




BATHROOM DESIGN IDEAS

Designed according to The Danish Building and Property Agency's guidelines for accessible design 2014, Building Regulations BR10 and BR08 and ISO 21542:2012.

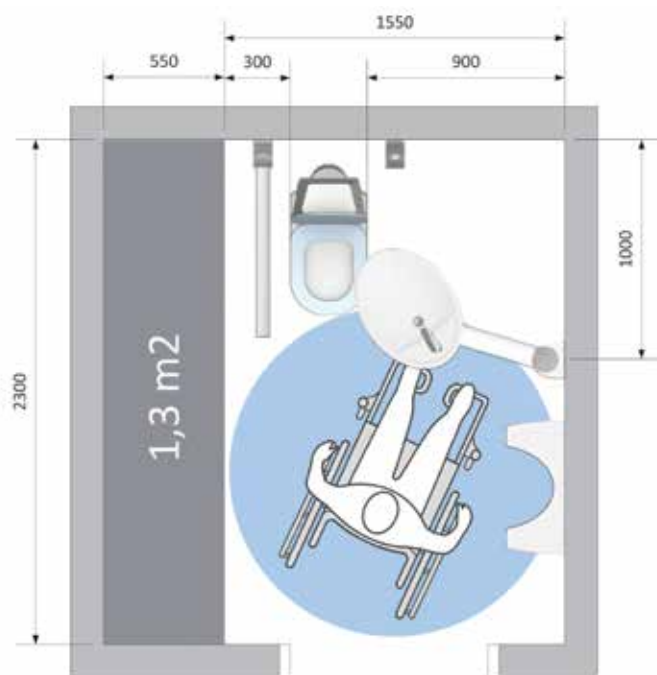


Toilet for self-sufficient wheelchair users

With Ropox solutions the bathroom area can be reduced by 1.3 m² compared to a standard bathroom.

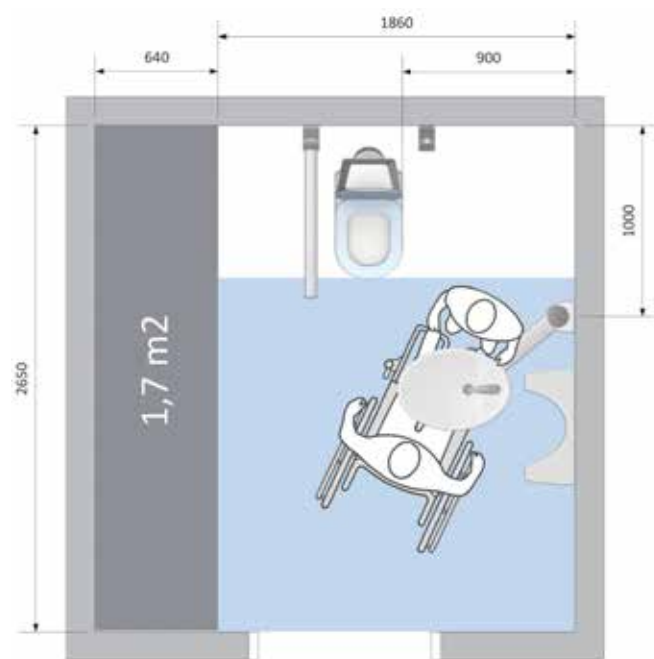


Toilet for wheelchair users who need a carer's assistance

With Ropox solutions the bathroom area can be reduced by 1.7 m² compared to a standard toilet bathroom.

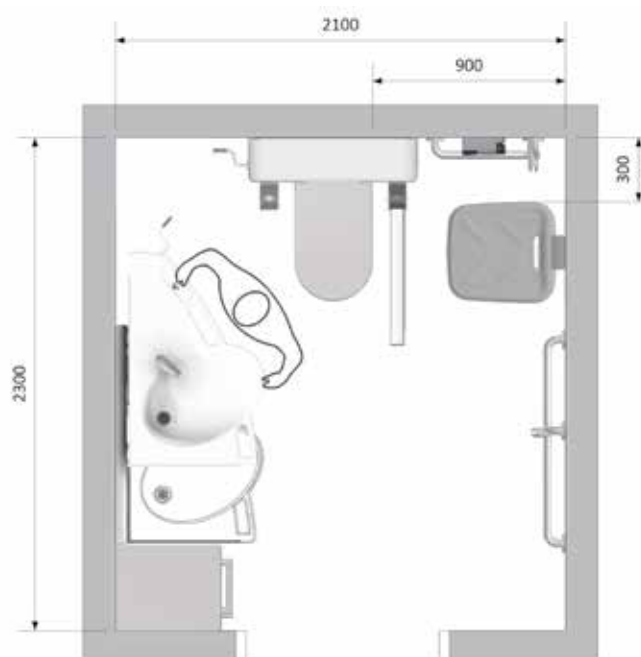

Toilet for walking-impaired users

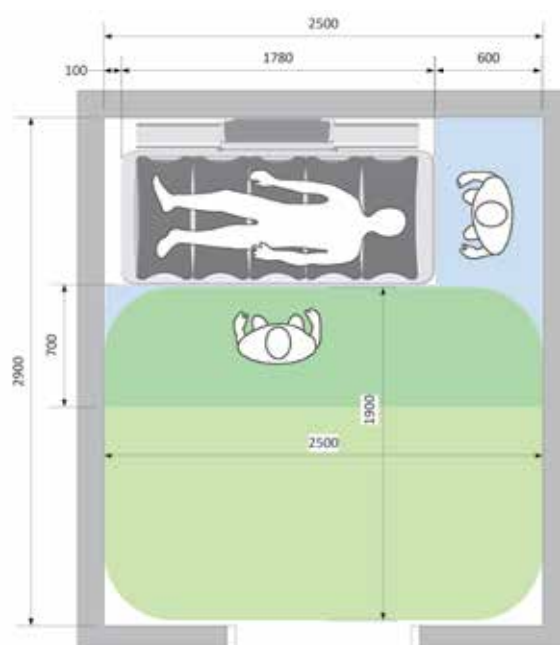

Shower/changing bed for users who need a carer's assistance in institutions

THE VARIANTS ARE AVAILABLE IN OUR DATA SHEETS OR ON WWW.ROPOX.COM

SWING WASHBASIN CONCEPT



The bathroom provides optimal space utilisation and better conditions for wheelchair users and walking-impaired. The users can use the washbasin while seated and minimise the risk of falling injuries. Thanks to the moveable washbasin, wheelchair users will have more space to manoeuvre the wheelchair in the room. The flexible washbasin also provides the opportunity to design complete bathrooms in less space.

SUPPORT WASHBASIN CONCEPT



This bathroom interior is the ideal solution for users who do not need a carer's assistance when washing hands, using the toilet, and showering. The washbasin is height-adjustable, sideways slidable and easy to clean. The solid and integrated grab rails offer the user safety and minimise the risk of falling injuries. The toilet support arms are designed with two integrated heights which provide great support when using the toilet.

ABOUT ROPOX

Freedom and confidence to be independent

People with disabilities deserve a better life with more self-esteem, quality, and the freedom to safely look after themselves. This philosophy has been our goal since we set up our business back in 1962.

We are amongst the world's leading specialists and producers of assistive devices and furniture for the disabled as well as equipment for ergo- and physiotherapy and for rehabilitation. Our products are recognised and appreciated around the world for their high quality, flexibility, and accessibility.

Ropox has 60 employees and is represented in four continents of the world.

For more than 50 years we have created a wide range of products for people with disabilities. Today - thanks to our long experience - we have a unique know-how about accessibility and work environments.

For more information, contact Ropox or one of our representatives at www.ropox.com



ROPOX TABLES

BEAUTIFUL HEIGHT-
ADJUSTABLE TABLES FOR
INDIVIDUAL NEEDS



ROPOX KITCHENS

FLEXIBLE AND
HEIGHT-ADJUSTABLE
KITCHENS



ROPOX BATHROOMS

INNOVATIVE AND
EXCITING PRODUCTS
IN A BEAUTIFUL
COHESION



ROPOX NURSING TABLES

SAFE AND FLEXIBLE
SOLUTIONS IN AN
ERGONOMIC DESIGN



Bathrooms of The Future - Data Sheets

”Makes it easy
to reach the washbasin ”



ROPOX SWINGLINE

A FLEXIBLE WASHBASIN THAT MAKES IT EASY TO WASH HANDS

The swing washbasin is recommended for both seated and standing users. You can adjust the height and rotate it 180°, and you can pull it to the user.

Walking-impaired users are particularly content with this washbasin because it can be pulled to the toilet. This means that they do not have to get up to use it, but instead, they can safely wash their hands while seated on the toilet.

The swing washbasin is also very practical for wheel-chair users. You can move the basin when required, e.g. push the washbasin to the corner when you need extra room for turning the wheelchair.

The swing washbasins with a lifting unit can easily be adjusted in height when needed. The simple design without any dirt traps makes it easy to clean.


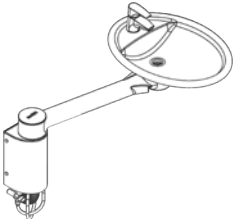
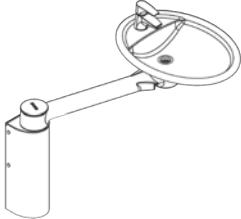
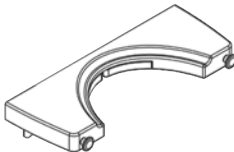



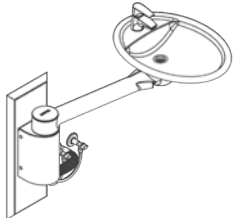
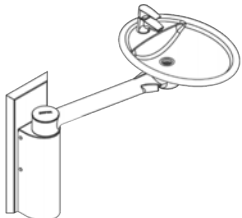
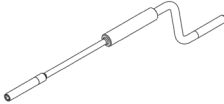
ROPOX SWINGLINE


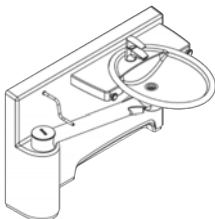
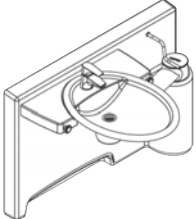
Height adjustment: 15 cm without Dock-In
20 cm with Dock-In


Maximum load: 300 kg (max user load 200 kg
in Dock-In and 120 kg without
Dock-In cf. DS/EN 17966:2016)

VARIANT/MODEL

WASHBASINS – FIXED HEIGHT 		ACCESSORY
		
Visible installation Item no.: 40-41110	Hidden installation Item no.: 40-41120	Accessory: Fixed height dock-in Item no.: 40-40076


WASHBASINS – HEIGHT ADJUSTABLE WITHOUT DOCK-IN 		ACCESSORY
		
Visible installation Item no.: 40-41130	Hidden installation Item no.: 40-41140	Accessory: Handle Item no.: 40-42170

WASHBASINS – ADJUSTABLE IN HEIGHT WITH DOCK-IN 	
	
Left hand model Item no.: 40-40072	Right hand model Item no.: 40-40071

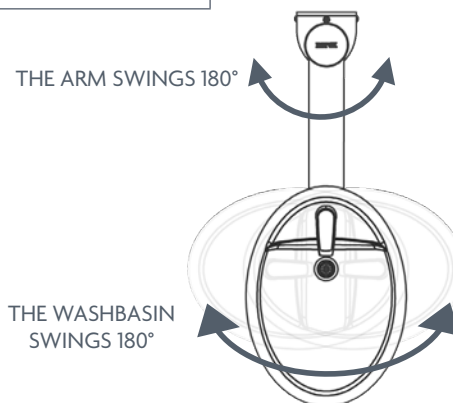


ACCESSORIES

Plug with overflow
Item no.: 40-40935

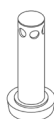


Accessory kit without wall
bracket/with wall bracket
Item no.: 40-41145/40-41150

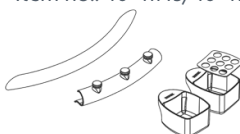

Extended lever
mixer tap
Item no.: 40-44042



Accessory tray
Item no.: 40-41146

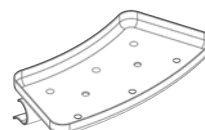

Trap
Item no.: 40-41170



Please note that all parts are suitable for dish washer/Autoclave.

NB! Find more information about measurements, technical data, installation interfaces and much more under the product and in the instruction manuals on our website.

The items are produced according to the following Standard: EN/EN17966-2016.

ROPOX A/S · Ringstedgade 221 · 4700 Naestved, Denmark · Phone: +45 5575 0500 · info@ropox.com · www.ropox.com

ROPOX™
BETTER WAYS TO BETTER DAYS

No. 1962241 0218

“With the right aids,
daily life becomes easier”



ROPOX SUPPORTLINE

A ROBUST WASHBASIN THAT CAN BE ADJUSTED TO MATCH YOUR NEEDS

A support washbasin is especially suitable for walking-impaired because you can use the integrated, ergonomic grab rails as support and as help when getting up or sitting down.













You can adjust the height of the washbasin to suit the physical capabilities of the individual user, and it is easy to slide the unit sideways allowing you to place the basin where you need it the most.

It is easy to keep the washbasin clean due to the cleaning-friendly design, and even though the basin is very robust and stable, it still has a simple and elegant look.

The solid, integrated grab rails ensure safety to lean on and use as a support when you move around in the room.

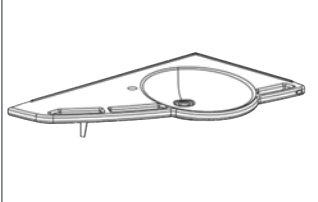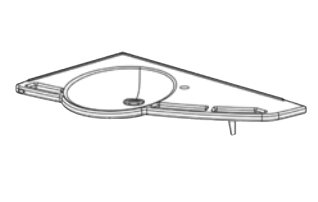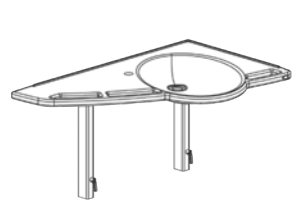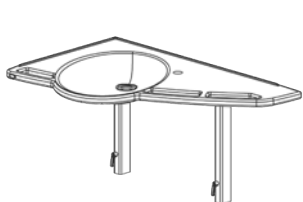

VARIANT/MODEL

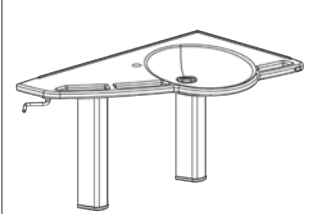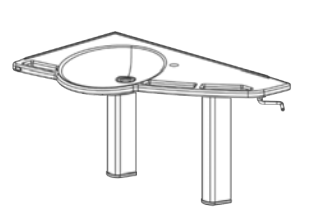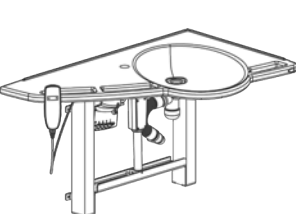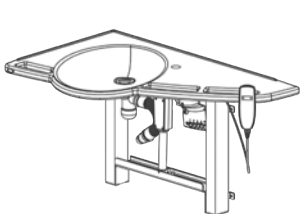WASHBASIN FIXED – FIXED HEIGHT 		WASHBASIN BASIC – PERIODIC ADJUSTMENT 	
			
Left hand model Item no.: 40-44010	Right hand model Item no.: 40-44011	Left hand model Item no.: 40-44012	Right hand model Item no.: 40-44013
WASHBASIN MANUAL – ADJUSTMENT WITH HANDLE 		WASHBASIN ELECTRIC – ELECTRICAL ADJUSTMENT 	
			
Left hand model Item no.: 40-44014	Right hand model Item no.: 40-44015	Left hand model Item no.: 40-44016	Right hand model Item no.: 40-44017


Variant Washbasin Electric include Plumbing kit.

ACCESSORIES

Plug with overflow
Item no.: 40-40935



Plumbing kit
Item no.: 40-44020 (Manual, Basic & Fixed)
Item no.: 40-44034 (Electric)
Item no.: 40-44031+32 (Manual)

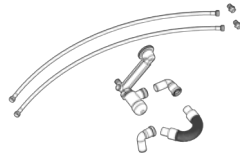


System for sideways sliding
Item no.: 40-44030 (Manual & Electric)



Special plumbing kit is required,
please contact Ropox.

Cover
Item no.: 40-44023+24 (Electric)
Item no.: 40-44027+28 (Manual)
Item no.: 40-44021+22 (Basic)

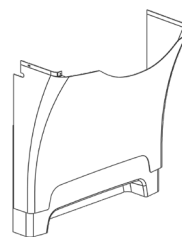

Fixed press pad
Item no.: 40-44026



For optimum choice of
mixer tap, please contact
Ropox.



Please note that all parts are suitable for dish washer/Autoclave.

NB! Find more information about measurements, technical data, installation interfaces and much more under the product and in the instruction manuals on our website.



ROPOX VANITYLINE

A STYLISH AND USER-FRIENDLY WASHBASIN

With a VanityLine washbasin you do not compromise on either design or user-friendliness because it has a modern and elegant look and at the same time, it is easy to use for people with limited physical capability.

There is plenty of room under the washbasin allowing easy access for wheelchair users. The bowl itself is only 120 mm high and the standard tap has a good height which secures that all users can reach the tap and get their hands under the jet without difficulty.

The washbasin has electrical or manual height adjustment. The electrical control switch is located on the front of the washbasin. This makes it easy to reach, and you can decide whether you prefer the switch on the left or right side of the washbasin.

VanityLine is available in three different sizes and two different models with the washbasin on the left side or right side of the unit because it is the little things that determine whether or not a sink is just right for a bathroom.

ROPOX VANITYLINE











Height adjustment:

68-98 cm

Max. user weight:

200 kg
(DS/EN17966-2016)

VARIANT/MODEL

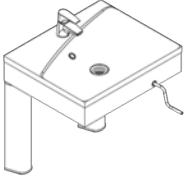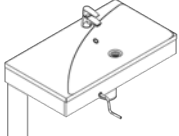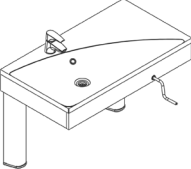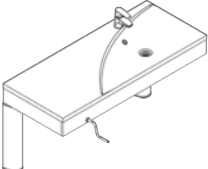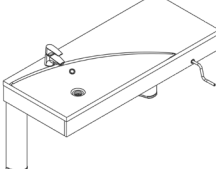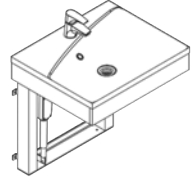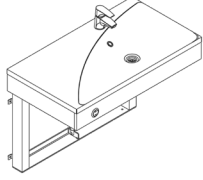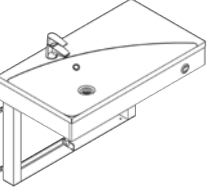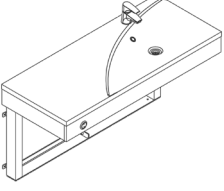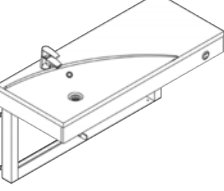WASHBASIN - MANUAL				
				
60 cm Item no.: 40-14871	90 cm Right Item no.: 40-14872	90 cm Left Item no.: 40-14873	120 cm Right Item no.: 40-14874	120 cm Left Item no.: 40-14875
WASHBASIN - ELECTRIC				
				
60 cm Item no.: 40-14851	90 cm Right Item no.: 40-14852	90 cm Left Item no.: 40-14853	120 cm Right Item no.: 40-14854	120 cm Left Item no.: 40-14855

Mixer tap and plumbing kit is included with the washbasins.





ACCESSORIES

Safety stop

60 cm Item no.: 40-14857
90 cm Item no.: 40-14858
120 cm Item no.: 40-17859

Cover

60 cm Item no.: 40-14865
90 cm Item no.: 40-14866
120 cm Item no.: 40-14867

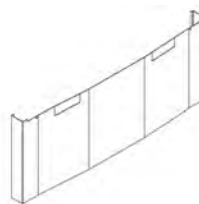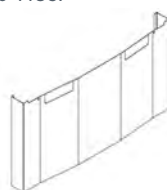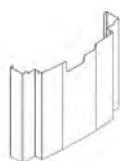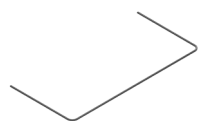

Hand control without safety strip
incl. hook
Item no.: 40-14868

Hand control with safety strip
incl. hook
Item no.: 40-14869

Wireless control, IR
Item no.: 30-72002

Plug with overflow
Item no.: 40-40935



NB! Find more information about measurements, technical data, installation interfaces and much more under the product and in the instruction manuals on our website.

The items are produced according to the following Standard: DS/ISO 17966-2016

ROPOX A/S · Ringstedgade 221 · 4700 Naestved, Denmark · Phone: +45 5575 0500 · info@ropox.com · www.ropox.com

ROPOXTM
BETTER WAYS TO BETTER DAYS

No. 1962237 0218

”A quality washbasin for private homes and institutions”



ROPOX STANDARDLINE

WHEN YOU WANT A HEIGHT-ADJUSTABLE WASHBASIN THAT WILL LAST FOR YEARS

The StandardLine is an excellent choice if you are looking for a functional quality washbasin, that is suitable for both private homes, institutions and hospitals.

The height is adjusted electrically, and you can decide if the control switch should be placed on the right or left side of the washbasin. That is one of the small details that can make the basin fit your individual needs in the best possible way.

There are three types of washbasins to choose from so you can be sure to get the model that fit the users best:

- Standard: A simple and stylish washbasin for users without any special needs
- Support: A washbasin with good access and integrated handles that increases safety for users with limited physical capacity
- Hospital: Porcelain washbasin with an inwardly curved front for hospitals

No matter which StandardLine model you choose, you can be sure to get a washbasin that will last for years and is easy to clean as well.



ROPOX STANDARDLINE



Height adjustment:

30 cm

Max. user weight:

200 kg
(DS/EN17966-2016)

VARIANT/MODEL

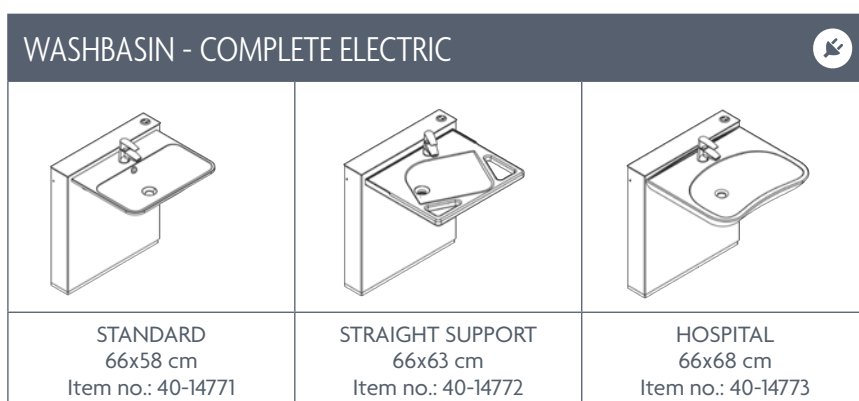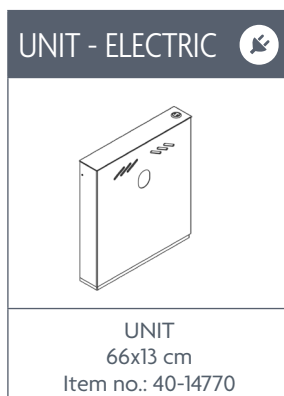



Available with or without wash basin.
Mixer tap and plumbing kit is included with the washbasins.

ACCESSORIES

Plumbing kit
Item no.: 40-14776

Wireless hand control, IR
Item no.: 30-72002

Plug with overflow
Item no.: 40-40935

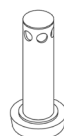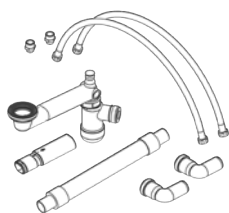

NB! Find more information about measurements, technical data, installation interfaces and much more under the product and in the instruction manuals on our website.

The items are produced according to the following Standard: DS/ISO 17966-2016

ROPOX A/S · Ringstedgade 221 · 4700 Naestved, Denmark · Phone: +45 5575 0500 · info@ropox.com · www.ropox.com

ROPOXTM
BETTER WAYS TO BETTER DAYS

No. 1962239 0218

”The washbasin that will
fit into any interior ”



ROPOX ADAPTLINE

A SMALL AND ELEGANT SOLUTION

AdaptLine is the obvious choice when you are looking to replace an old washbasin with a new one as it has a small wash unit and it can easily be adapted to existing water installations and drains.



The size also makes it fit into almost any environment which is an advantage when you furnish an interior where a washbasin is to be included.

You can adjust the height manually with a crank, and the design makes the unit easy to clean which allows you to keep a high level of hygiene.

AdaptLine is the flexible and elegant solution when you need a smaller washbasin which will fit most interiors.




VARIANT/MODEL

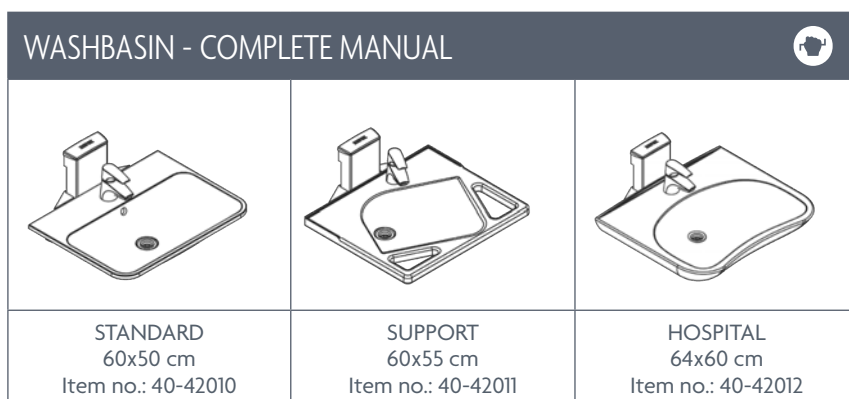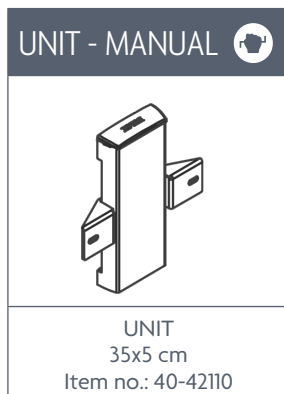


Available with or without wash basin.
Mixer tap and plumbing kit is included with the washbasins.

ACCESSORIES


Flexible axle
Item no.: 40-42170

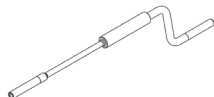

Plumbing kit
Item no.: 40-25731

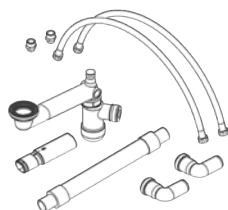

Plug with overflow
Item no.: 40-40935

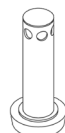

NB! Find more information about measurements, technical data, installation interfaces and much more under the product and in the instruction manuals on our website.

”The optimum washbasin for both seated and standing users”



ROPOX SLIMLINE

WHEN YOU PREFER WASHBASIN AND MIRROR IN A COMBINED, SLIM AND ELEGANT SOLUTION

The SlimLine washbasin combines mirror and basin in one unit, and the mirror is therefore always at the right height no matter if you are sitting in a wheelchair or standing up. You adjust the height with a push on the control switch which is always within reach as it is located right under the mirror.

There are three types of washbasins to choose from, and each is particularly suited for specific users' needs:

- Standard: A simple and stylish washbasin for users without any special needs
- Support: A washbasin with good access and integrated handles that increases safety for users with limited physical capability
- Hospital: Porcelain washbasin with an inwardly curved front for hospitals

If a user's needs change over time, it is easy to replace the washbasin with another model since they are quite straightforward to install. The round corners of the SlimLine models minimize bacteria growth allowing you to keep a high hygiene level.



ROPOX SLIMLINE



Height adjustment:

70-100 cm

Max. user weight:

200 kg
(DS/EN17966-2016)

VARIANT/MODEL

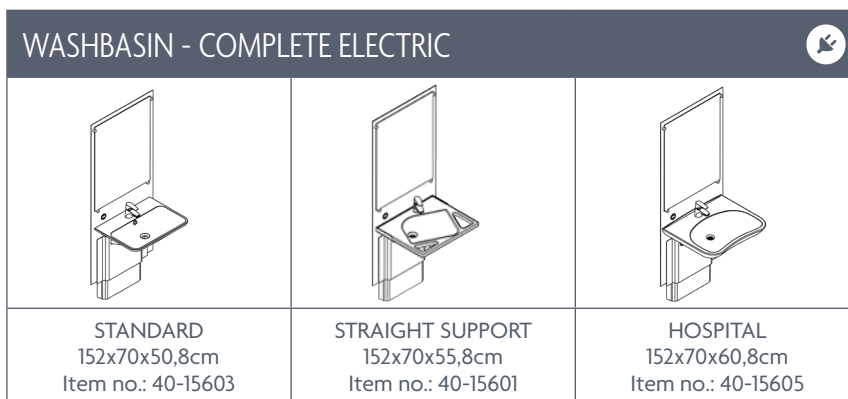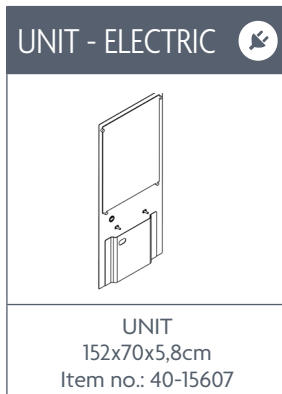

Available with or without wash basin.

Mixer tap and plumbing kit as well as cover for installation is included with the washbasins.

ACCESSORIES




Plumbing kit
Item no.: 40-15612

Fixed light length 600mm
Item no.: 40-44182

Cover
Item no.: 40-15610

Hand control
Item no.: 40-15630

Plug with overflow
Item no.: 40-40935

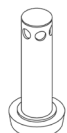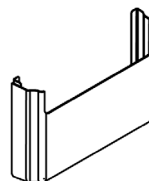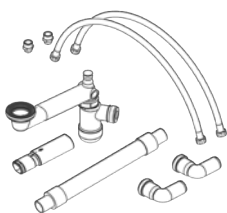

NB! Find more information about measurements, technical data, installation interfaces and much more under the product and in the instruction manuals on our website.

The items are produced according to the following Standard: DS/ISO 17966-2016

ROPOX A/S · Ringstedgade 221 · 4700 Naestved, Denmark · Phone: +45 5575 0500 · info@ropox.com · www.ropox.com

ROPOXTM
BETTER WAYS TO BETTER DAYS

No. 1962238 0218

”Doing more personal care
without assistance”



ROPOX TOILET LIFTER

GET ON AND OFF THE TOILET MORE EASILY AND SAFELY

Most people would like to take care of their personal hygiene without assistance from others. The toilet lifter is one of the aids that can enable you to become more self-sufficient in getting on and off the toilet because it makes it easier and more effortless to sit down and stand up.

The toilet lifter lowers and lifts the user to the best sitting or standing position, and it is straightforward to set to the optimum height for both user and helper.

The round edges are gentle to the skin, and you do not get scratches, and the cleaning-friendly design makes the toilet lifter easy to keep clean ensuring a high level of hygiene.






ROPOX TOILET LIFTER

Height adjustment: Electric / Manual 40 cm
Basic 20 cm

Maximum load: 600 kg (max user load 400 kg
cf. DS/EN 17966:2016)

VARIANT/MODEL

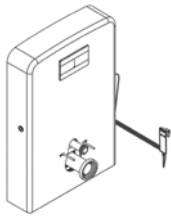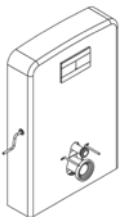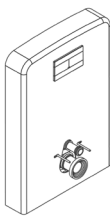
TOILET LIFTER		
		
Electric Item no.: 40-45020	Manual Item no.: 40-45030	Basic Item no.: 40-45040

Toilet lifter unit in bottom position: Model Electric/Manual 118 x 79 x 20,5 cm and Model Basic 118 x 79 x 16,5 cm.

Dual flushing system at 3 and 6 litres.

ACCESSORIES


Toilet supports with height adjustable unit. See data sheet for arm supports for more options.

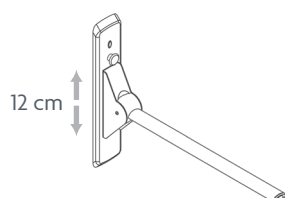


For optimum choice of toilet, please contact Ropox.



Upgrade kit (Manual to Electric)
Item no.: 40-45102

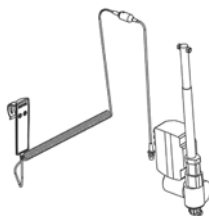

Long backrest
Item no.: 40-45100

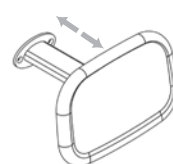

Short backrest
Item no.: 40-45101



NB! Find more information about measurements, technical data, installation interfaces and much more under the product and in the instruction manuals on our website.

The items are produced according to the following Standard: EN/EN17966-2016.

ROPOX A/S · Ringstedgade 221 · 4700 Næstved, Denmark · Phone: +45 5575 0500 · info@ropox.com · www.ropox.com

ROPOXTM
BETTER WAYS TO BETTER DAYS

No. 1962244 0218

”More independence
with your personal care”



ROPOX TOILET SUPPORT ARMS

GET SOMETHING TO LEAN ON

Toilet support arms are a great aid if you need to get on and off the toilet safely. Their ergonomic shapes provide a comfortable grip, and the cleaning-friendly surfaces optimize hygiene.

Because they are designed with two heights and an anti-slip surface, you have a firm grip from both standing and seated position. If you need a little extra support for getting up and sitting down, you can mount the standing/sitting attachments.

The support arms can be adjusted in height and at different angles to suit the individual user's needs.

If you need extra room in the bathroom, the support arms can be folded up to vertical position.






ROPOX TOILET SUPPORT ARMS

Height adjustment: 12 cm

Maximum load: 400 kg (see max. user load below cf. DS/EN 66:2016)



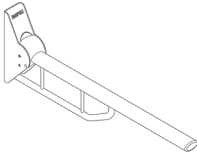
VARIANT/MODEL

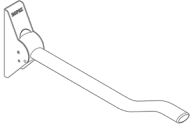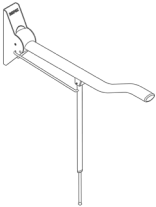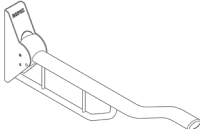WAVE - 90 CM		
		
80 kg Item no.: 40-40115	135 kg Item no.: 40-40125	190 kg Item no.: 40-40145

WAVE - 76 CM

110 kg Item no.: 40-40151

Can also be used for Ropox Toilet Lifter

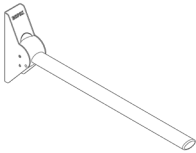STRAIGHT - 90 CM		
		
80 kg Item no.: 40-40110	135 kg Item no.: 40-40120	190 kg Item no.: 40-40140


STRAIGHT - 76 CM

110 kg/190 kg Item no.: 40-40150/40-40152


Can also be used for Ropox Toilet Lifter

ACCESSORIES

Height adjustment
(adjusted with tool)
Item no.: 40-40906

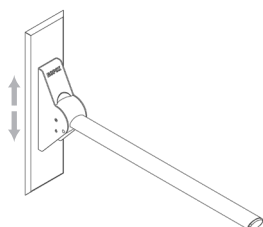

Height adjustment
(adjusted without tool)
Item no.: 40-40907

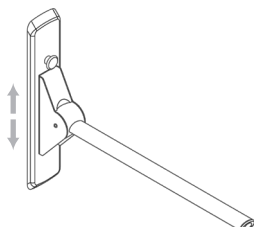

Toilet roll holder
Item no.: 40-40910



Standing/Sitting attachment
Item no.: 40-40925



NB! Find more information about measurements, technical data, installation interfaces and much more under the product and in the instruction manuals on our website.

The items are produced according to the following Standard: EN/EN17966-2016.

ROPOX A/S · Ringstedgade 221 · 4700 Naestved, Denmark · Phone: +45 5575 0500 · info@ropox.com · www.ropox.com

ROPOXTM
BETTER WAYS TO BETTER DAYS

No. 1962243 0218



”When you want
to spare your efforts”

ROPOX SHOWER SEAT

ALLOWS YOU TO TAKE A REST IN THE SHOWER

A shower seat enables you to sit safely and comfortably while showering.

The seat is foldable, therefore it takes up very little space when it is not in use. It has a very soft padding and is made in a hard-wearing, insulating material which does not feel cold on the skin.

The seat has a large surface area and a solid grabbing edge so that you can have a comfortable rest in safe surroundings.

You can easily adjust the seat's height to meet individual needs, and if you would like more support, you can choose a backrest to lean on either with or without arm supports.







ROPOX SHOWER SEAT

Height
adjustment: 12,5 cm

Maximum load: 500 kg (max 200 kg with leg
and 150 kg without leg
cf. DS/EN 17966:2016)

VARIANT/MODEL

SHOWER SEATS	
	
With leg Item no.: 40-43011	Without leg Item no.: 40-43016

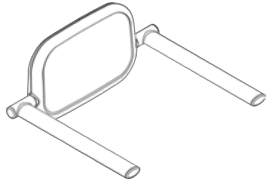
BACKRESTS	
	
With arm support Item no.: 40-43025	Without arm support Item no.: 40-43030

The shower seat and the arm supports are foldable which provides more space in the bathroom.

ACCESSORIES

Height adjustable unit for
shower seat
Item no.: 40-4304

Height adjustable unit for shower
seat and backrest
Item no.: 40-43043

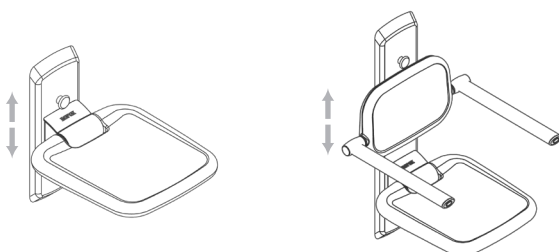

NB! Find more information about measurements, technical data, installation interfaces and much more under the product and in the instruction manuals on our website.

The items are produced according to the following Standard: EN/EN17966-2016.

ROPOX A/S · Ringstedgade 221 · 4700 Naestved, Denmark · Phone: +45 5575 0500 · info@ropox.com · www.ropox.com

ROPOXTM
BETTER WAYS TO BETTER DAYS

No. 1962245 0218

”Become more self-sufficient
with a little extra support”



ROPOX GRAB AND SHOWER RAILS

SAFELY MANOEUVRE WITH GRAB RAILS

Grab and shower rails are robust and stable aids for the independent user in situations where they need a little extra support.

The grab rails are available in a number of lengths and shapes enabling you to design a bathroom with the right aids at the right place for the individual user.

Due to the design and size of the grab rails, you will have a firm and stable grip. We have designed them without any dirt traps, so that you can keep a high hygiene level.






The grab rails for adjusting the height of the hand shower can be used with just one hand making it very easy to use.






ROPOX GRAB AND SHOWER RAILS

Maximum load: 400 kg
(see max. user load below cf.
DS/EN 17966:2016)

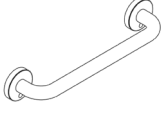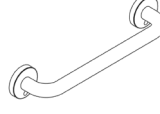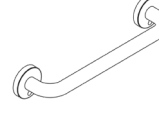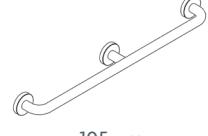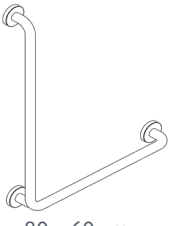VARIANT/MODEL


GRAB RAILS				
				
40 cm	60 cm	80 cm	105 cm	80 x 60 cm
200 kg Item no.: 40-44108 white Item no.: 40-44118 grey	160 kg Item no.: 40-44109 white Item no.: 40-44119 grey	120 kg Item no.: 40-44110 white Item no.: 40-44120 grey	200 kg Item no.: 40-44126 white Item no.: 40-44127 grey	160 kg Item no.: 40-44128 white Item no.: 40-44129 grey

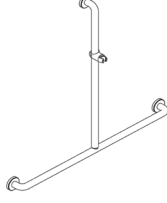SHOWER RAILS WITH SHOWER HOLDER		
		
100 cm	100 x 45 cm	113,5 x 42,5 x 52,5 cm
250 kg Item no.: 40-44111 white Item no.: 40-44121 grey	250 kg Item no.: 40-44112 white Item no.: 40-44122 grey	250 kg Item no.: 40-44113 white Item no.: 40-44123 grey

The solid grab and shower rails are available in many lengths and shapes.
All have anti-bacterial surfaces.
The L- and T-shaped models can be reversed if needed.

ACCESSORY

Shower shelf
Item no.: 40-44130

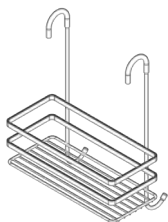

NB! Find more information about measurements, technical data, installation interfaces and much more under the product and in the instruction manuals on our website.

The items are produced according to the following Standard: EN/EN17966-2016.

ROPOX A/S · Ringstedgade 221 · 4700 Naestved, Denmark · Phone: +45 5575 0500 · info@ropox.com · www.ropox.com

ROPOXTM
BETTER WAYS TO BETTER DAYS

No. 1962247 0218

”Making it safe and comfortable to take a shower”



ROPOX SHOWER/CHANGING BED

SAFE AND EASY TO USE

The shower/changing bed gives the user optimum comfort and safety, and the helper gets the best possible working position. This provides security and satisfaction for both parties.

The bed is made of double-sided slats. One side of the slat curves slightly downwards ensuring a safe position for the person bathing. The other side curves slightly upwards making it easy for a helper to turn the user on the side when using the bed for changing clothes.

It is easy to adjust the height of the bed with the hand control, and the bed can quickly be adjusted to the best position, e.g. for a wheelchair so transferring a user onto the bed is smooth and easy.

The shower/changing bed is cleaning-friendly, and can be folded up when not in use to save space in the bathroom.









ROPOX SHOWER/CHANGING BED


Height adjustment: 70 cm (30-100 cm) above floor

Maximum load: 220 kg cf. DS/EN 17966:2016

VARIANT/MODEL

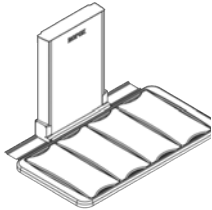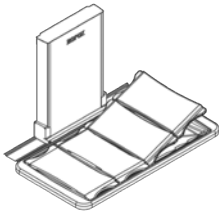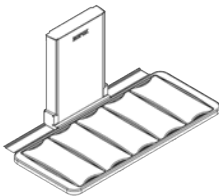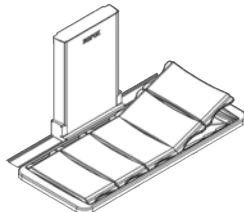SHOWER/CHANGING BED - 146 CM		SHOWER/CHANGING BED - 178 CM	
			
Without back support Item no.: 40-25023	With back support Item no.: 40-25073	Without back support Item no.: 40-25026	With back support Item no.: 40-25076

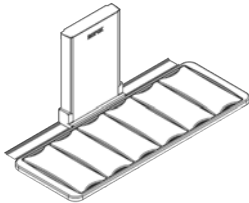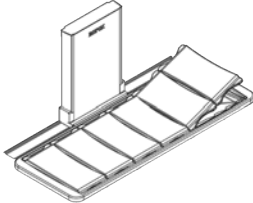SHOWER/CHANGING BED - 210 CM	
	
Without back support Item no.: 40-25029	With back support Item no.: 40-25079

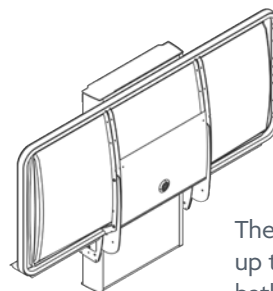


The bed can be folded up to save space in the bathroom.


The shower/changing bed includes an electrical hand control for easy height adjustment.
The lying area is 18 cm shorter than the length of the shower/changing bed.
Bed and end guards can be mounted if needed. The transparent bed guard keeps the water spillage to a minimum.

ACCESSORIES


Back support
Item no.: 40-25031

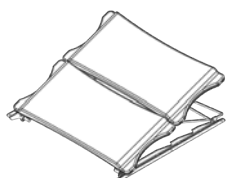


End guard
Item no.: 40-25038

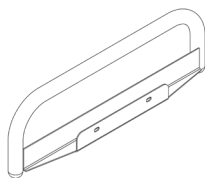

Bed guard
Item no.: 40-25036

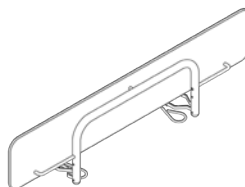

Headrest
Item no.: 40-25090

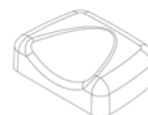

NB! Find more information about measurements, technical data, installation interfaces and much more under the product and in the instruction manuals on our website.

The items are produced according to the following Standard: EN/EN17966-2016.

ROPOX A/S · Ringstedgade 221 · 4700 Naestved, Denmark · Phone: +45 5575 0500 · info@ropox.com · www.ropox.com

ROPOXTM
BETTER WAYS TO BETTER DAYS

No. 1962246 0218

”A bathtub everyone
can use”



ROPOX BATHTUB

EASY AND GENTLE BATHING

When you need extra support from a carer to take a bath, a bathtub that can be adjusted in height makes it easy and comfortable to get in and out of the bath.

It is easy to adjust the tub to the optimum working height for the helpers. The helpers can also use a mobile lift for helping a person to the bathtub as there is the necessary room under the tub for the lifting device. This provides safety and satisfaction for both carer and user.

You adjust the height with the hand control and getting in and out of the bath matches the user's physical capabilities, this also provides the optimum working position for the helper.

A shower bed cover can be fastened to the top of the tub adding to the comfort when bathing.

The bathtub is made in a simple design which is easy to clean.




ROPOX BATHTUB

Height adjustment: 30 cm (61-91 cm) above floor


Maximum load: 350 kg (incl. user and water)
cf. DS/ISO 17966:2016 350

VARIANT/MODEL

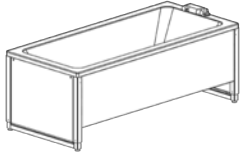BATHTUB - 160X70 CM 	BATHTUB - 170X75 CM 	BATHTUB - 180X80 CM 
		
Inside measurements 142x57 cm Item no.: 40-14156	Inside measurements 152x62 cm Item no.: 40-14157	Inside measurements 164x67 cm Item no.: 40-14158

The tub is 40 cm deep on all models.

Bathtub incl. hand shower, mixer tap and flexible plumbing kit.

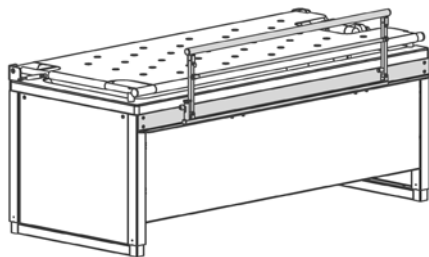


The bathtub mounted with a foldable shower bed and bed guard.


The bathtub is easy to heightadjust with the electrical hand control which is included.

ACCESSORIES


Shower bed for mounting on the tub. Contact Ropox for different models.

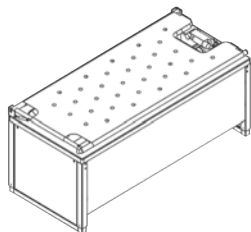


Support for hand shower
Item no.: 40-14120
To be ordered with the tub

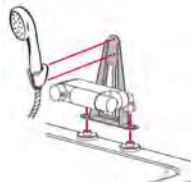

Bed guard for bathtub
Item no.: 40-14186 for 160 cm
Item no.: 40-14187 for 170 cm
Item no.: 40-14188 for 180 cm

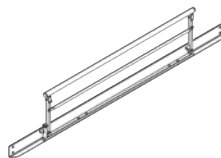

Brake wheels incl. fittings (4 pcs.)
Item no.: 40-14111 for 160 cm
Item no.: 40-14112 for 170 cm
Item no.: 40-14113 for 180 cm

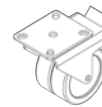

Safety stop (two sides)
Item no.: 40-14216 for 160 cm
Item no.: 40-14217 for 170 cm
Item no.: 40-14218 for 180 cm



Safety stop (one side)
Item no.: 40-14206 for 160 cm
Item no.: 40-14207 for 170 cm
Item no.: 40-14208 for 180 cm



NB! Find more information about measurements, technical data, installation interfaces and much more under the product and in the instruction manuals on our website.

The items are produced according to the following Standard: EN/EN17966-2016.

ROPOX A/S · Ringstedgade 221 · 4700 Naestved, Denmark · Phone: +45 5575 0500 · info@ropox.com · www.ropox.com

ROPOXTM
BETTER WAYS TO BETTER DAYS

No. 1962248 0218